Gebrauchsanweisung	2	Upute za upotrebu	25	Petunjuk penggunaan	47
Instructions for Use	3	Návod na použitie	27	使用说明	49
Istruzioni per l'uso	5	Navodila za uporabo	28	사용 설명서	50
Mode d'emploi	7	Uputstvo za upotrebu	30	使用方法	51
Brugsanvisning	8	Οδηγίες χρήσης	32	คำแนะนำในการใช้งาน	53
Bruksanvisning	10	Kullanım talimatı	33	გამოყენების ინსტრუქცია	54
Käyttöohje	12	Használati útmutató	35	تعليمات الاستخدام	56
Gebruiksaanwijzing	13	Инструкции за употреба	37	دستورالعملهای استفاده	58
Bruksanvisning	15	Instrucțiuni de utilizare	39	הנחיות שימוש	59
Instruções de utilização	18	Lietošanas instrukcija	40		
Инструкция по применению	20	Naudojimo instrukcija	42		
Instrukcja użytkowania	22	Kasutusjuhend	44		
Návod k použití	23	Інструкція з користування	45		

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Essential COMFORTABLE  
Essential THERMOREGULATING  
Essential CLASSICAL  
Style SEMITRSPARENT  
Style PATTERN  
Active MASCULINE  
Dynaven CLASSIC  
Dynaven CLASSIC Sleeve

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## English

### Intended purpose

Compression stockings are aids for compression therapy. The products are intended for long-term therapy in chronic venous insufficiency as well as for supporting lymph drainage. The product is intended for multiple use and for application on the legs (lower leg, thigh, entire leg).

### The effects of medical compression stockings

Compression therapy is used to improve quality of life in chronic venous diseases, for the prevention and therapy of venous edemas and venous skin changes, as well as for ulcer prevention and ulcer therapy of the legs. Carefully controlled pressure on the tissue and veins improves the blood return in the legs and increases lymphatic flow. At the same time, it improves microcirculation and therefore the supply to the tissue.

### Target group

The intended target group includes patients who have been prescribed a stocking for individual compression therapy by the responsible physician or trained health professional. The target group also includes user groups from health care professions who apply the products to patients.

## Indications & contraindications

### Indications

#### *Chronic venous diseases*

Improvement of venous symptoms; improvement of quality of life in chronic venous diseases; prevention and treatment of venous edemas as well as venous skin changes; eczema and pigmentation; lipodermatosclerosis and atrophie blanche; prevention and treatment of leg ulcers and pain reduction in venous leg ulcers; varicose veins; initial phase after varicose vein therapy; functional venous insufficiency; venous malformations

#### *Venous thrombotic diseases*

Superficial venous thrombosis; deep venous thrombosis of the leg; status post thrombosis; post-thrombotic syndrome; thrombosis prophylaxis in mobile patients

#### *Edemas*

Lymphedema; edema in pregnancy; occupational edema; drug-induced edema; post-traumatic and postoperative edema; postoperative reperfusion edema; idiopathic cyclic edema; lipedema up to Stage II; congestive states as a result of immobility

#### *Other Indications*

Obesity with functional venous insufficiency; inflammatory dermatosis of the legs; nausea, dizziness and congestive conditions in pregnancy; status post burns; scar treatment

### Contraindications

Medical compression stockings must not be worn in the following cases:

- Advanced peripheral arterial occlusive disease (ABPI <0.6, ankle blood pressure <60 mmHg, toe blood pressure <30 mmHg, or TcPO<sub>2</sub> <20 mmHg instep).
- Severe cardiac insufficiency (NYHA III–IV)
- Septic phlebitis
- Phlegmasia coerulea dolens
- Compression of an existing arterial bypass (epifascial)

Relative contraindications that require a particular balancing of benefits and risks:

- Allergy against compression material
- Inflammatory processes (compression products to reduce inflammation, pain or edema in leg erysipelas or cellulitis should only be used in combination with antibacterial treatment)
- PAOD with ABPI between 0.6 and 0.9 and/or ankle pressure between 60 and 90 mmHg
- Skin diseases with massive exudation
- Severe sensory disturbances of the extremity
- Advanced peripheral neuropathy (e.g. in diabetes mellitus)
- Primary chronic polyarthritis

### Risks and Side Effects

Medical compression stockings can cause skin necrosis and pressure damage to the peripheral nerves if handled improperly. Itching, scaling, and signs of inflammation may occur under the compression aids in cases of sensitive skin, so adequate skin care under the compression aids is advised.

### Special instructions

A manifest lymphedema requires medically prescribed complex decongestive treatment. Regular medical check-ups are required to maintain successful therapy. If acute pain or skin irritation occurs while wearing, immediately consult your physician.

Wash compression products before first use.

Do not cut loose ends on the insides of the stocking. Protect the compression stockings from ointments, oils, and grease to preserve their service life.

Active MASCULINE contains a yarn treated with zinc oxide.

Compression stockings must not be worn over open wounds.

For products with grip tops: regularly ventilate the skin.

Noncompliance with this information may limit safety and effectiveness of the product.

All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

If the product is damaged, its medical effectiveness is no longer guaranteed.

### Application

We recommend wearing special gloves to make it easier to put the stockings on and take them off. Sharp fingernails and rings can cause defects.

### Donning

1. The best time is when rising in the morning, when the swelling in your legs has subsided. For easier application, make sure that your legs are dry.
2. Slip the stocking evenly over your heel. Do not pull, but spread the fabric evenly over the leg using gloves. While doing so, check for a proper fit at the toe and heel. For the Dynaven CLASSIC Sleeve stocking, check that the grip top is above the ankle.
3. To protect the seam and the grip top, carefully grasp the knitted fabric and not the seam when donning the compression stocking.
4. If it needs repositioning, work the fabric upwards towards the knee with the flat of the hand and spread evenly throughout.

### Correct fit

- Knee high stockings: 2 finger widths below the back of the knee
- Half-thigh stockings: up to the middle of the thigh
- High stockings: up to 3 finger widths below the buttocks/crotch

For advice on donning and doffing our compression stockings, we recommend watching the videos available at [www.sigvaris.com/videos](http://www.sigvaris.com/videos).

### Doffing

1. Take hold of the stockings at the upper edge with gloves and pull downwards towards the heel.
2. Work the fabric over the heel with the flat of the hand.

## Donning and doffing aids

For easier donning and doffing of our compression stockings we recommend the donning aids MELANY, DOFF N' DONNER, magnide, or sim-slide.

For more information, contact your specialist dealer or visit our homepage: [www.sigvaris.com](http://www.sigvaris.com)

## Storage, wearing period and service life

Unless otherwise prescribed by your physician, wear your compression stocking daily from morning to evening for optimal therapy results. Wearing and washing your compression stocking every day may cause the medical pressure gradient and elasticity of your compression stocking to diminish. We guarantee maximum effect for six months of wearing for series and customized stockings.

The shelf life date of the series stockings is indicated on the packaging using the symbol of an hourglass. After this date, the stockings can be used for the wearing time recommended above.

## Care and storage

Follow the instructions for care on the packaging and/or the textile labels.

Wash inside a laundry net. The stocking must be stored at room temperature and in a dry place away from light. Keep away from direct heat.

## Material abbreviations

CO	cotton
EL	elastane
PA	polyamide

## Applies to the following products

Essential COMFORTABLE

Essential THERMOREGULATING

Essential CLASSICAL

Style SEMITRSPARENT

Style PATTERN

Active MASCULINE

Dynaven CLASSIC

Dynaven CLASSIC Sleeve

## Disposal

There are no special requirements for the disposal of medical compression stockings. These can be disposed of as residual or household waste. Please comply with local regulations for the disposal of textiles.

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SIGVARIS and SIGVARIS GROUP are registered trademarks of SIGVARIS AG,
© 2024 Copyright by SIGVARIS AG, CH-9014 St. Gallen/Switzerland
SIGVARIS GmbH, Dr.-Karl-Lenz-Strasse 35, 87700 Memmingen, **Germany**
SIGVARIS S.r.l., Via Galileo Galilei 2 / 2, 31057 Silea (TV), **Italy**
SIGVARIS S.A., ul. Powidzka 50, Gutowo Małe, 62-300, Września, **Poland**
SIGVARIS (Australia) Pty. Ltd., 8/53 Stanley Street, West Melbourne VIC 3003, **Australia**