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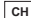
EN **Instructions for use** MOBILIS EPIBAND Elbow strap

Intended use:
Medical device for treatment in the event of acute and/or chronic symptoms.

Target group:
Device designed for persons showing the symptoms described in the indications.

Indications:
Tennis elbow / golfer's elbow (epicondylopathy).

Contraindications:
Diseases, soreness and skin injuries at the sites of product application, in particular acute inflammation; open and exuding wounds; raised scars with edema, redness and excessive warming; disturbed lymphatic drainage, including edema of the soft tissue of unclear origin distal to the applied product; post-traumatic and post-operative conditions requiring complete immobilisation.

Storage:
This product must be stored in its original packaging, at room temperature and in a dry, dark environment. Keep away from direct heat (e.g. heating, sunlight) and moisture.

Precautions:

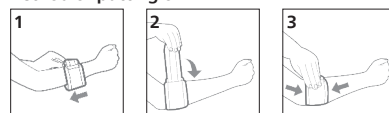
- Only use this product on unbroken skin. Do not wear this product over wounds, damaged skin or skin affected by a disease.
- In the case of persistent or intensified pain, allergic reaction, skin irritation or any other adverse effects related to the use of the product, discontinue use and consult a doctor.
- Consultation with a medical professional may be required to ensure proper use of this product.
- Correct sizing is very important in order for this product to function properly.
- Check the product before each use for signs of wear or deterioration. This product can only function properly if it is completely intact.
- Do not wear the product 24 hours a day, unless prescribed by a doctor.
- The use of ointments, oils, creams and similar substances can have a negative effect on the product's properties, decrease its durability and reduce its effectiveness.

Information on initial preparation of the product:

It is recommended to wash the product before the first use.

Care: Wash in the washing bag; close Velcro fastenings before washing. 

Method of putting on:

All serious incidents that occur when using the product shall be reported to the manufacturer and the competent supervisory authority.

 SIGVARIS S.A., ul. Mazowiecka 11 lok. 49, 00-052 Warszawa, Poland
  SIGVARIS AG, Gröblistrasse 8, CH-9014 St. Gallen/Switzerland

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SIGVARIS, SIGVARIS GROUP and MOBILIS are registered trademarks of SIGVARIS AG, CH-9014 St. Gallen / Switzerland in many countries worldwide. © 2024 Copyright by SIGVARIS AG, CH-9014 St. Gallen/Switzerland V5 / 01-2024