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DE

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User Manual

EN

Please read the entire user manual carefully before assembling, adjusting, or using the Actimo® Light rollator for the first time. Keep the user manual in a safe place for future reference.

Intended Use

The Actimo® Light rollator is designed to assist individuals with limited mobility and movement due to illness or surgery. The Actimo® Light is suitable for both indoor and outdoor use. The rollator features swivel front wheels, a seat, a cane holder, and a bag for transporting items. The seat allows the user to take rest breaks. The rollator can be folded compactly by lifting the seat and sliding the rollator together. The handle height is adjustable.

Indications

The Actimo® Light can be used for mild balance issues and limited mobility due to weakness or reduced mobility. The padded backrest provides safe seating.

Contraindications

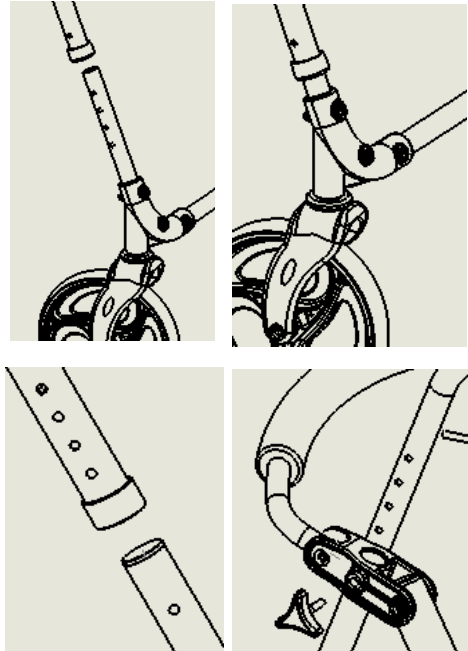
The use of the rollator is not recommended for individuals with:

- Sensory impairments
- Severe balance problems
- Loss of both arms
- Joint damage or joint contractures in both arms
- Reduced or inadequate vision

Assembly


Your dealer will assist you with the assembly.

1. Remove all parts from the box, check for completeness and damages, and have them ready.
2. Press down on the seat board to unfold the rollator; the upper crossbar should audibly lock into place.
3. The rollator is now secure and unfolded. Insert the frame part with the front wheels into the rollator frame and adjust the height using the silver push buttons.
4. Insert the two rear wheels into the openings of the rear frame part, pressing the silver push buttons. Adjust the desired height by snapping the silver buttons into the designated openings. Make sure the brake cables are not pinched.
5. Stand the rollator up and insert the handles with the brakes from the top into the frame up to the desired height. Secure them with the wing nuts.
6. To attach the net, place the open loops around the rollator frame below the seat and fasten them with the snap buttons..

Braking and Locking Brakes

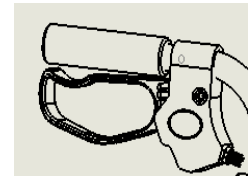- To brake during rollator use, pull both brake handles evenly upwards. Adjust the braking force intensity based on speed and terrain.
- To engage the brake lock, push the brake handles down with your palms until they audibly and tangibly click into place.
- To disengage the brake lock, pull both handles evenly upwards until they release from the lock.

Folding and Unfolding the Rollator

- Before folding the rollator, empty the net.
- Stand behind the rollator, make sure the brakes are engaged. Lift the seat upwards and slide the rollator together.
- To unfold, stand behind the rollator, release the backrest, push the seat downwards, and ensure the seat locks audibly. You can release the brakes, and the rollator is ready to use.

Safety Instructions

- Use the rollator only when it's in perfect condition and make sure the seat board is securely in place.
- Do not use a defective rollator!
- The ends of the handles must be parallel to the ground. Check the secure fit of the wing nuts and the correct height adjustment before use.
- Test the brakes before each use and always use both the right and left brakes simultaneously.
- Pay attention to tire wear. Heavily worn tires can affect the rollator's braking. Replace damaged parts through your dealer.
- The rollator is not a wheelchair; do not allow yourself to be pushed while sitting on the seat board.
- Always sit with the brakes engaged on the seat board.
- The rollator must not be used as a seat in vehicles such as trains or buses.
- Use the rollator only on even surfaces – never on stairs or high edges! To ensure stability and prevent accidents, all four wheels must have contact with the ground during use.
- Always hold both handles to maintain balance.
- Do not carry bags on the handles. There's a risk of tipping!
- Do not overload the rollator (max. load capacity: 5 kg).
- The maximum user weight is 136 kg.
- Do not make any structural modifications to the rollator without consulting your dealer or Rehaforum Medical GmbH.

Cleaning

Clean and disinfect the rollator regularly, for example, daily. Before passing the rollator to another person, thorough cleaning and disinfection are mandatory. Use a mild soapy solution without abrasive additives and a soft cloth. Disinfection can be done with commercially available products according to the manufacturer's instructions. Do not use abrasive or corrosive cleaning agents or high-pressure steam cleaners. Ensure that dirt does not accumulate on the tires or brakes. This can be removed with a soft brush and warm water. Also, make sure that cleaning fluid does not enter the frame tubes to prevent corrosion.

Maintenance

The Actimo® Light rollator should be serviced annually by your dealer. The stability of the frame, the strength of the mountable parts, and the tightness of all screws must be checked and adjusted if necessary. All screws must be tightened. Worn out, damaged, or broken parts must be replaced promptly.

Functional error:

If you notice a decrease in brake power, wheel stiffness, unsteady wheel operation, or other issues such as rust on the frame, contact your dealer. Do not use a defective or damaged rollator.

Reuse

The rollator is suitable for reuse. Before handing it to another user, the rollator must undergo inspection, cleaning, disinfection, and maintenance by the dealer.

If parts of the rollator are damaged or destroyed, they must be replaced with new ones. It is recommended to replace the wheels with each user change. All loose screws and nuts must be tightened.

The product must be in perfect condition before reuse.

It is forbidden to reuse the rollator if irreparable damage to the rollator, such as a bent frame, is detected.

The user manual should be provided with each change of user.

Scope of delivery

Actimo® Light including cane holder
Shopping bag/net
User manual

Technical Data

Weight: 7.9kg
Folded size: approx. 98 - 118 cm x 61 cm x 38 cm
Width (frame): 61 cm
Height (adjustable handles): approx. 85-99 cm
Depth: approx. 73 cm
Seat width: 36 cm
Width between handles: 54 cm
Width of the handles: 30 mm
Seat height (including upholstery): approx. 53 - 61 cm
Tire size: 8"
Turning circle: 94 cm
Recommended height: 160-190 cm
Maximum load: 136 kg
Max. load/bag: 5 kg
Material frame: aluminum (anodised)
Seat cushion material: polyurethane (PU)/polypropylene (PP)
Backrest material: ethylene vinyl acetate (EVA)
Material wheels: PP/PVC

Storage/Transport

Protect the rollator from dust and moisture and store it in a safe place to prevent damage to the frame and wheels. For safe transport, the rollator must be folded; lift the seat using the recessed handle after removing the net bag. Protect your rollator from strong heat sources.

Disposal

Do not dispose of the rollator with regular household waste. Please contact your dealer or take the rollator to your local disposal site..

Service Life

If the rollator is used within its intended purpose and maintained, inspected, and serviced according to the instructions, a service life of five years can be expected for the Actimo® Light rollator. However, the service life can vary significantly depending on the treatment and care of the product. A particularly careful handling of the product can extend its service life, while extreme and improper handling can shorten it.

The determination of the service life by Rehaforum Medical GmbH does not constitute an additional warranty.

Warranty

The warranty period is 24 months and covers all product defects that can be attributed to manufacturing or material faults.

The warranty claim does not apply if the rollator shows signs of repairs or interventions by unauthorized dealers or if non-authorized accessories are installed without prior written consent from Rehaforum Medical GmbH.

Reporting/Incidents

All serious incidents related to the product must be reported to the manufacturer and the relevant authority of the member state in which the user and/or patient is located.

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Fastlæggelsen af levetiden af Rehaforum Medical GmbH udgør ikke en yderligere garanti.
Rehaforum Medical GmbH, Daimlerstraße 12a, DE-25337 Elmshorn